



Preparation Steps to Start Ablation:

1. Turn Power switches to **ON** position on front and back of machine.
2. Use the **Left Arrows** to scroll to desired antenna and push the **M Button**.
3. Use the **Left Arrows** to adjust to desired Temperature.
4. Use the **Right Arrows** to adjust to the desired total Time.
5. When antenna cable is connected correctly, screen will say **READY** on bottom right.
6. Press **Yellow Start/Power Button** to start ablation.
7. Press **Yellow Start/Power Button** to stop/pause ablation. Press the same button to continue ablation or **M Button** to abort ablation.